# NITRO SL

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This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

## Table of contents

|                                         |    |
|-----------------------------------------|----|
| Preface .....                           | 11 |
| Declaration of conformity .....         | 11 |
| Service life .....                      | 11 |
| Disposal .....                          | 11 |
| Identification plate .....              | 11 |
| Reuse .....                             | 11 |
| Warranty .....                          | 11 |
| Parts of the product .....              | 12 |
| Intended purpose .....                  | 13 |
| Indication .....                        | 13 |
| Scope of delivery .....                 | 13 |
| Safety instructions .....               | 13 |
| Assembly .....                          | 13 |
| Unfolding .....                         | 14 |
| Assembly of accessories .....           | 14 |
| Height adjustment of push handles ..... | 14 |
| Adjusting the brakes .....              | 14 |
| Braking   Parking .....                 | 14 |
| Adjusting the backrest .....            | 15 |
| Cane holder .....                       | 15 |
| Sitting .....                           | 15 |
| Curb Climber .....                      | 15 |
| Folding   Transporting   Stowing .....  | 15 |
| Cleaning   Maintenance .....            | 15 |
| Technical data .....                    | 16 |

## Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Accessible instructions for use in easy and/or simple language are not available from the manufacturer.

## Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

## Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee

## Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal centre.

## Identification plate

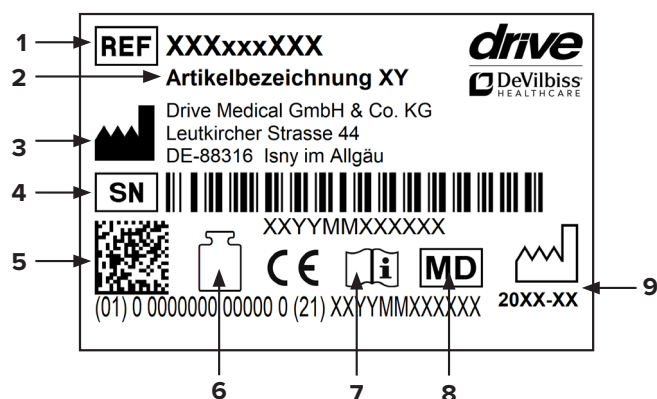1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

## Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at [www.drivedevilbiss.de](http://www.drivedevilbiss.de) and [www.drivedevilbiss.co.uk](http://www.drivedevilbiss.co.uk) (UK specialists).

## Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

### EXCLUSION OF WARRANTY

Parts subject to wear include (among others): wheels, brakes and handles.

## Parts of the product

**NITRO SL**

- |                                 |                  |                     |
|---------------------------------|------------------|---------------------|
| 1. Handle                       | 7. Front castor  | 12. Curb climber    |
| 2. Push handle                  | 8. Cane holder   | 13. Front reflector |
| 3. Brake                        | 9. Bag           | 14. Rear reflector  |
| 4. Fixing screw for push handle | 10. Back support | 15. Side reflector  |
| 5. Seat                         | 11. Bowden cable |                     |
| 6. Rear wheel                   |                  |                     |

## Intended purpose

Walkers are used to support safe walking and, depending on the design, can also be used to relieve pressure. Four-wheel walking frames are products made from tubular frames which have four solid rubber wheels. These walking frames are designed such that the user always moves between the contact points or wheels. These walking frames have braking systems which operate using Bowden cables operated by handgrips. These walking frames can be folded for transportation and the handgrips are height-adjustable to suit different heights.

## Indication

### INDICATION:

Walking aid for outdoor and indoor use to compensate for or improve limited walking ability. Insured parties are to be provided with walking aids with the intention of compensating for or at least improving the user's limited walking ability (expansion of radius of activity or mobility).

Persons with limitations of hand/arm functions require particular adjustments/designs of handles or handrests/armrests. Movable walking aids may not be suitable for persons with balance disorders; in individual cases, this should be tested out. Where appropriate, an attendant must be present. The rollator is to be used only for normal, everyday activities.

### CONTRAINDICATION:

The rollator is not suitable for use in the case of: perception disorders, severe loss of balance, inability to sit down.

## Scope of delivery

|     |                     |
|-----|---------------------|
| 1 x | Rollator            |
| 1 x | Bag                 |
| 1 x | Back support        |
| 1 x | Cane holder         |
| 1 x | Instruction for Use |

## Safety instructions

- Use the rollator only as a walking aid!
- Use the rollator only on an even and solid surface.
- Rollator must not be used as a seat in vehicles, such as a bus or train!
- It is prohibited to stand on the rollator or to transport a person or animal on the rollator.
- Take note of the permitted maximum weight of the user.
- This information can be found in the technical data in these instructions for use as well as on the identification plate.
- Use the bag only to transport objects and do not overload it! Max. load: 5.0 kg
- Use on escalators and the like is expressly prohibited!
- Do not hang any bags on the handle – risk of tipping!
- On steep surfaces, use the service brake (pull the brake lever upwards).

- Leaning excessively on the handles can cause the rollator to tip backwards.
- When folding/unfolding the rollator, clothing and body parts can be pinched.
- Protect the rollator from direct sunlight. Product parts can become hot and cause skin injuries if touched.
- Prior to each use, ensure that your rollator is intact and all adjustable parts are properly secured. Check that the brakes are working properly!
- Before using the rollator, ensure that the folding guard is properly locked.
- Check the screws on the rollator regularly and tighten, if necessary.
- The braking effect can be impaired by wear and tear of the tyres. For this reason, check the braking effect and tyres regularly.
- The rollator should not be carried by the seat, removable parts or the brake cables.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the MHRA or competent authority of your member state.
- In order to use the product correctly, it must be adjusted to the individual size of the user. This should be carried out by a trained person.
- **Risk of entrapment!** Take care of your hands, fingers and clothing when operating the folding mechanism while folding and unfolding the rollator. There is a risk of injury!

## Assembly

- Take all the parts out of the box.
- Unfold the frame.
- Insert the castors, making sure that you position the forks pointing outwards when the rollator is in motion.
- "R" label = right side
- "L" label = left side

**Caution: Make sure that the two push-buttons are fully and properly engaged.**

Straighten the frame and insert the rear wheels. Guide the rear wheels inside through the folding mechanism so that the bowden cables do not run outside of the frame. If the brake pad is rubbing on the wheel, take the assembly back off and unwind the cable a few turns. Adjust the cable using the screw situated at the base and/or on the handle side.

## Unfolding

Place the rollator on its wheels.

Now press both side rails of the seat section downward until they click into place (**pic. 1**).

The rollator is unfolded correctly when both locking hooks have clicked into place (**pic. 2**).

If it is difficult to engage the seat section while unfolding, this can be remedied by applying a small film of grease to the locking hooks.

## Assembly of accessories


The bag is hung by its loops from the ends of the frame tubing (**pic. 3**).

## Height adjustment of push handles

Insert push handles at least to the first cross hole into the frame tubing. Insert the fixing screw from the outside and tighten clockwise (**pic. 4**).


To change the height adjustment of the push handles, loosen the fixing screw by turning the star screw counterclockwise. Then lock the handle at the desired height, as described above (**pic. 5**).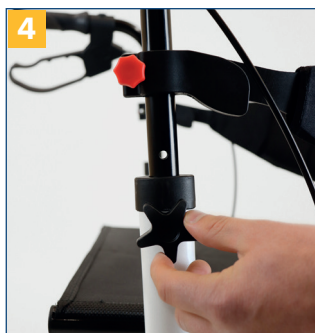

## Adjusting the brakes

If braking is poor or too hard, the brake can be adjusted either at the brake housing above (**pic. 6**) or using the adjusting screw on the Bowden cable below (**pic. 7**) to achieve the desired braking effect.

The brake is correctly adjusted when the brake lever as well as the brake wires are free of play and the wheels can be rotated without grinding noises when the brake is released.

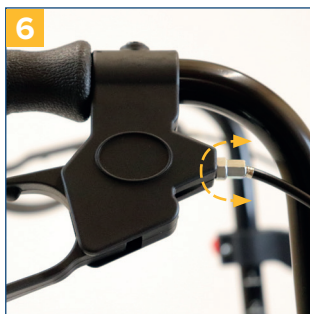## Braking | Parking

To brake when walking with the rollator, pull both brake handles upwards with your hands (**pic. 8**).

Always brake slowly while applying equal pressure with both hands.

To apply the brakes/park the rollator, press both brake handles downwards until they audibly click into place (**pic. 9**).

To release the brake, pull both brake handles upwards once again until they are released (brake handles are in the middle position).

The braking effect can be impaired by wear and tear on the tyres. For this reason, regularly check the braking effect.

## Adjusting the backrest

Position the sling by inserting the fixings on each side into the guides on the brackets situated at the level of the handles (pic. 10).

Adjust the supports to the height required.

**Caution:** Before each use, make sure that the sling has been inserted correctly and that the two brackets are fully tightened. Make sure that you fit the sling the right way (stitching facing outwards).

## Cane holder

Clip the upper cane holder below the yellow reflector to the frame. The lower cane holder part must be fixed on the lower frame part (pic. 11).

Make sure correct assembly. The cane holder can be mounted left or right.

## Sitting

Place the rollator on an even and solid surface. Then apply the brakes. The rollator must be stable and may not slip. Only sit upright; do not lean backwards (pic. 12)!

Seated position permitted only as shown and against the direction of movement (pic. 13)!

## Curb Climber


To negotiate kerbs or similar increases, please use the curb climber provided for this purpose. Make sure that you have a secure footing when using it. Press the tip of your foot on the curb climber so that the front wheels of the rollator lift up and push the rollator over the rise. Remove your foot from the curb climber as soon as you have overcome the obstacle.

**Caution:** Your balance may be affected when using the curb climber!

## Folding | Transporting | Stowing

To fold up the rollator, use the handle to pull the seat section upwards (pic. 14-15).

**Remove accessories, if necessary!**

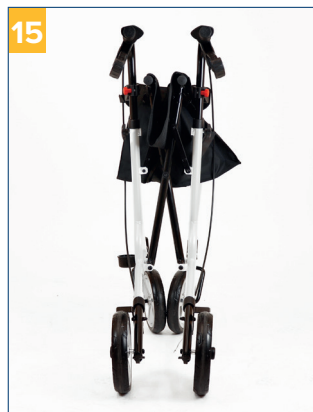## Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60° C for long periods of time (for example, use in a sauna, etc.)!
- In the event of defects, contact your specialist dealer from whom you purchased this product!
- Check the condition of the tyres regularly. They are “failsafe”, meaning that you do not have to check air pressure or fill them with air. Always check the tyres for tears or other damage, however.
- Corrosion-protected materials were used to manufacture the product. If, nevertheless, damage is detected with regard to corrosion, the rollator must no longer be used and the specialist dealer must be contacted immediately.

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**Drive Medical GmbH & Co. KG**
**Drive DeVilbiss Australia PTY LTD**