TSE 120 / TSE 120 Plus

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Table of contents

Preface.....	9
Declaration of conformity.....	9
Service life	9
Disposal.....	9
Identification plate.....	9
Reuse.....	9
Warranty	9
Intended purpose	10
Indication.....	10
Scope of delivery.....	10
Safety instructions.....	11
Use.....	11
Assembly.....	11
Height settings	13
Technical data.....	13
Cleaning Maintenance	13

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co.KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate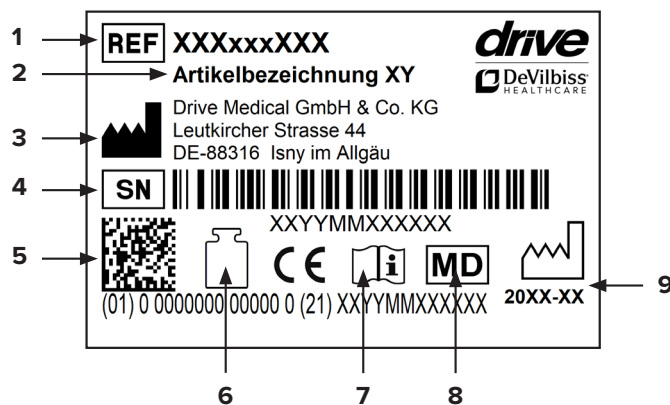

EN: 1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year/month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.





Accessories are available at www.drivedevilbiss.de or from your specialist dealer.

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.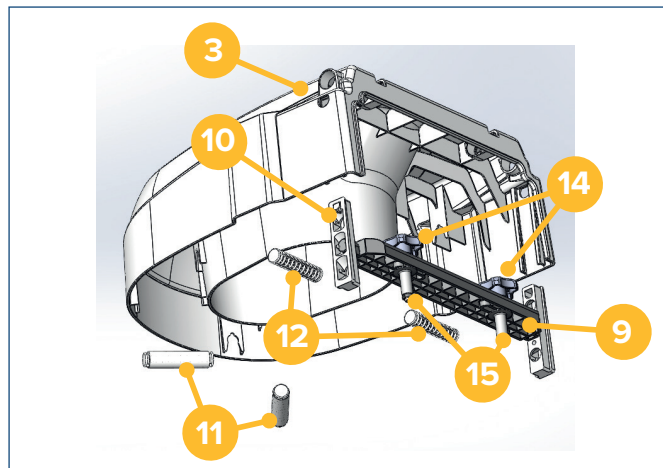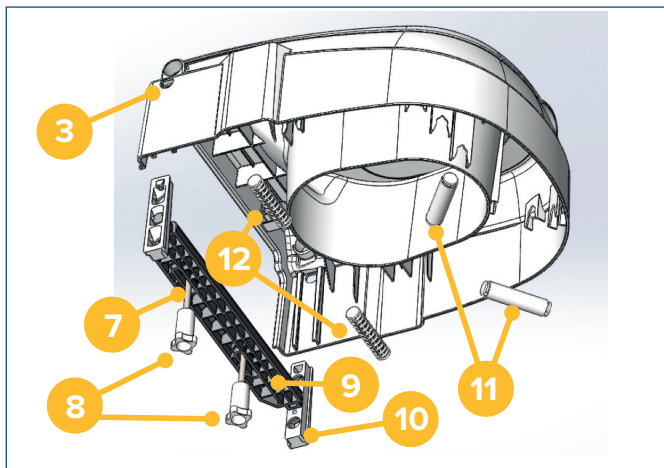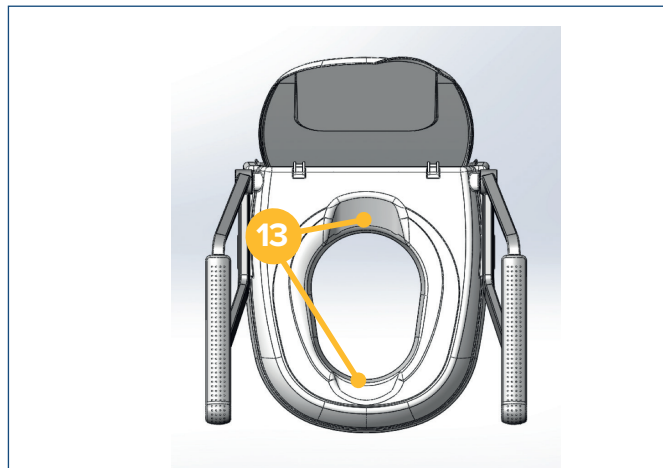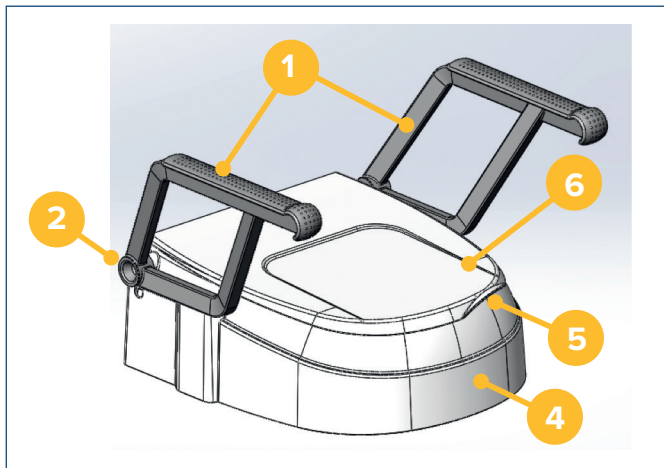

TSE 120

TSE 120 Plus

1. **Armrest**
2. **Armrest holder**
3. **Locking mechanism for adjustable holder**
4. **Body**
5. **Recessed grip (in the lid)**

6. **Lid**
7. **Fixing screw**
8. **Knurled nuts**
9. **Rear support**
10. **Adjustable holders**

11. **Front support**
12. **Braces**
13. **Hygiene openings**
14. **Star screw**
15. **Top-mounting pin**

Intended purpose

Toilet seats are placed over the existing toilet bowl or screwed onto the toilet and are used to raise the height of the toilet. Toilet seat raisers can be used with and without armrests. Toilet aids enable or increase independence when using the toilet and the related personal hygiene, particularly because they compensate for the impaired or missing functions of the motor system or muscular support structure.

Indication

INDICATION:

Use of the toilet hindered due to unadapted seat height through limited ability to sit down and stand up along with limited control of the trunk with functional disorder of the lower extremity with significantly restricted or fully absent movement of the joints and/or limited muscular strength of the lower extremity and/or damage to the lumbar spine, hip and/or knee and/or damage to the nervous system/central nervous system and/or damage to muscle function with limited or absent stability of the trunk.

CONTRAINDICATIONS:

The toilet seat raiser is to be used exclusively to raise a toilet seat and is therefore designed to fit onto a toilet bowl. Any use other than the use stated is not permitted. The toilet seat raiser is only designed for adults. Do not use as a child's toilet seat. With correct use, there are no contraindications.

Scope of delivery

1 x A	Body
1 x B	Support
2 x C	Adjustable holder
2 x D	Front support
2 x E	Brace
2 x F	Fixing screws (only for TSE 120)
2 x G	Knurled nuts (only for TSE 120)
2 x H	Star screw (only for TSE 120 Plus)
2 x I	Top-mounting pin (only for TSE 120 Plus)
2 x J	Armrest (not for 520500300 TSE 120 without armrest)
1 x	Instructions for use
1 x	Vinyl gloves

Safety instructions

- Do not use if there are any defects!
- Caution – risk of slipping/falling!
- The lid must not be used as a backrest in the opened position!
- The lid must not bear a load when closed (by sitting on it, for example)!
- Check the raised toilet seat regularly for a secure fit on the toilet bowl. Retighten the knurled nuts if necessary.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.

Use


The raised toilet seat TSE 120 is suitable for nearly all standardized or commercially available toilet bowls.

Assembly


Remove the seat and lid from the toilet bowl.
Assembly is carried out without tools.

TSE 120:

Place the rear of the support on the ceramic bowl (**B**) with the fixing screws (**F**). Screw in the knurled nuts (**G**) from below but do not tighten them yet. Align the rear of the support on the center of the ceramic bowl and tighten the knurled nuts (**photo 1**).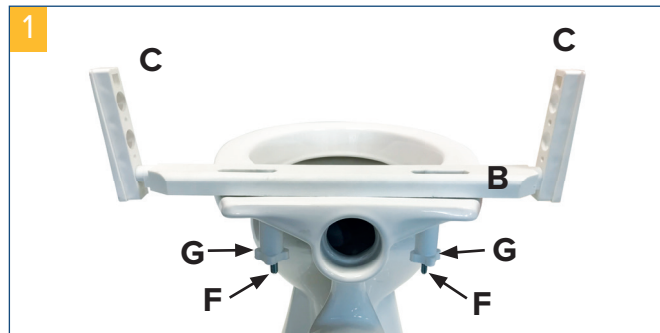


Insert the adjustable holders (**C**) at the desired height (8.5 / 12 / 15.5 cm) on the rear of the support (**B**) on the right and left (**photo 2**).

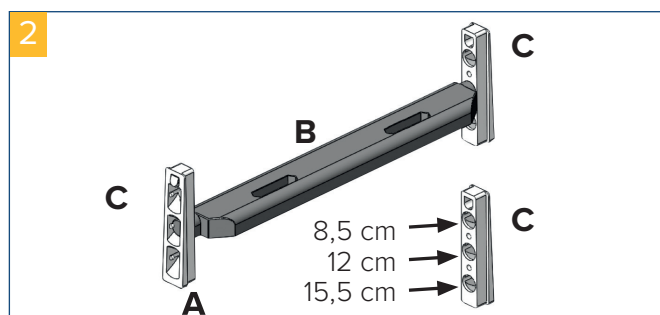
To adjust the height in the front, insert the supports (**D**) in the corresponding opening until they audibly click into place (**photo 3**).

- Take care that the front supports (**D**) on the right and left are each inserted at the same height.
- The front supports (**D**) can be set lower than the rear support (**B**).
- Insert the two braces (**E**) in the corresponding openings in the rear until they audibly click into place (**photo 3**).
- Insert the body of the raised toilet seat (**A**) into the adjustable holders (**C**). In doing so, the adjustable holders (**C**) must engage with the locking tabs (**L**) in the body (**A**) (**photo 4+5**).

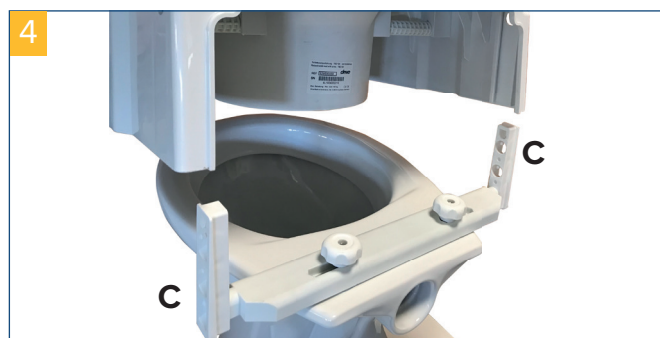

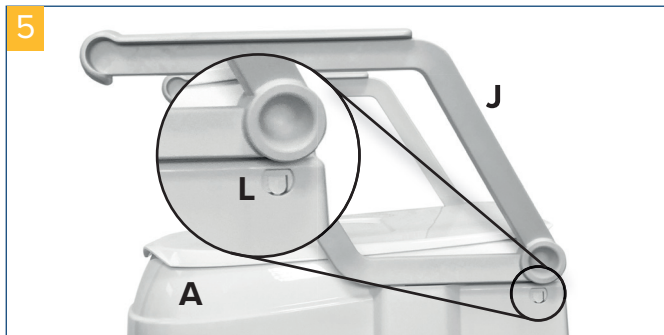The adjustable holders (C) can only be inserted in one direction into the raised toilet seat.

TSE 120 PLUS:

Push the top-mounting pin together (I) as shown in the photo and insert down into the mounting holes of the ceramic bowl (photo 6).

Place the rear of the support (B) on the ceramic bowl and screw in the star screws (H) from above but do not tighten them yet. Align the rear of the support (B) in the center on the ceramic bowl and tighten the star screws (H) (photo 7+8).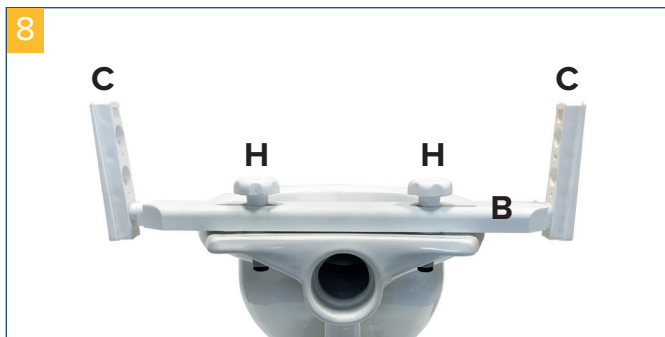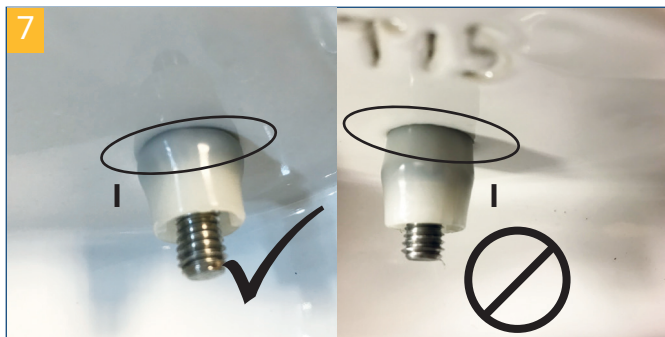

Insert the adjustable holders (C) at the desired height (8.5 / 12 / 15.5 cm) on the rear of the support (B) on the right and left (photo 2).

To adjust the height in the front, insert the supports (D) in the corresponding opening until they audibly click into place (photo 3).

- Take care that the front supports (D) on the right and left are each inserted at the same height.
- The front supports (D) can be set lower than the rear support (B).
- Insert the two braces (E) in the corresponding openings in the rear until they audibly click into place (photo 4).
- Insert body (A) of the raised toilet seat into the adjustable holders (C). In doing so, the adjustable holders (C) must engage with the locking tabs (L) in the body (A) (photo 4+5).

The adjustable holders (C) can only be inserted in one direction into the raised toilet seat.


LID ASSEMBLY:

The lid is preassembled at the time of delivery, but can be replaced if necessary. To do this, turn the body (A) of the raised toilet seat around and press the lid clips at the same time.


The lid can then be taken out of the holders. Clip the new lid on.

ARMREST ASSEMBLY:

Push the armrests (J) sideways into the armrest holders provided for this purpose (photo 9).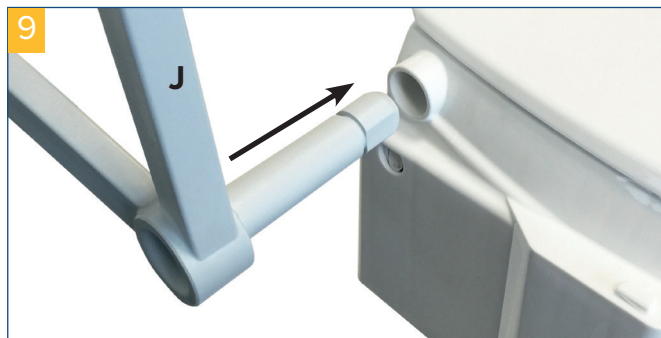

When doing so, the arrows on the armrest (J) and the armrest holder should match up (photo 10).

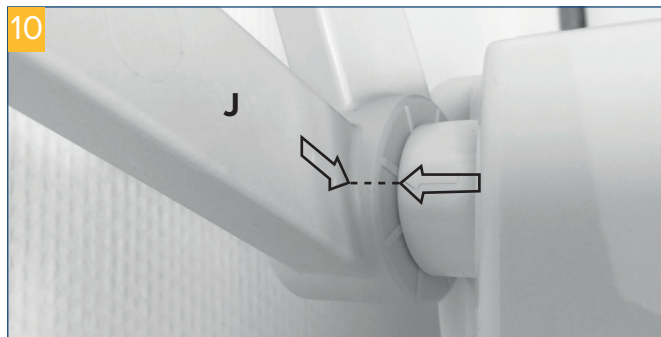Now push the armrests (J) in until they catch, and then fold them down.

Height settings

The raised toilet seat can optionally be adjusted using the adjustable holders **(C)** for an elevation of 8.5 cm, 12 cm or 15.5 cm.

- Slightly push in the locking tabs **(L)** on the right and left under the armrest holders and then lift the raised toilet seat.
- Pull out the adjustable holders **(C)** and adjust the desired height **(photo 10)**.
- Take care that the adjustable holders **(C)** on the right and left are each connected at the same height.
- To adjust the height in the front, insert the supports **(D)** in the corresponding opening until they audibly click into place.
- Take care that the front supports **(D)** on the right and left are each inserted at the same height.
- The front supports **(D)** can be set lower than the rear support **(B)**.
- Insert the body of the raised toilet seat **(A)** into the adjustable holders **(C)**. In doing so, the adjustable holders **(C)** must engage with the locking tabs **(L)** in the body **(A)**.
- The adjustable holders **(C)** can only be inserted in one direction into the raised toilet seat.

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60°C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.

Technical data

Product	TSE 120 without armrest	TSE 120 with armrest	TSE 120 PLUS
Article number	520500300	520500400	520500500
Overall length	49 cm	49 cm	49 cm
Overall width	46 cm	55 cm	55 cm
Overall height	19 cm	38 cm	38 cm
Opening length	27 cm	27 cm	27 cm
Opening width	20.5 cm	20.5 cm	20.5 cm
Width between armrests	-	46 cm	46 cm
Height adjustment	8.5 / 12 / 15.5 cm	8.5 / 12 / 15.5 cm	8.5 / 12 / 15.5 cm
Weight	2.6 kg	4.1 kg	4.1 kg
Max. load	120 kg	120 kg	120 kg
Color	white	white	white
Material	Plastic (PP / PA)	Plastic (PP / PA)	Plastic (PP / PA)

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Drive Medical GmbH & Co. KG